

**Instructions for the use  
and cleaning of the  
Arianna BABY-Vac Nasal Vacuum  
Revision No.6**

**Important:**

Clean unit before and after each use by washing it with hot soap water. Rinse with hot or warm water. Do not use any detergents, chemicals or abrasive material on this. Clean nose tip with soft pad or cotton swab moistened with ethylalcohol. Do not use any other chemicals.

**Warning!**

In the case of hemorrhage, infection, fever or nasal irritation please consult a doctor.

**Caution!**

- Use a vacuum that is rated between 800-1800 Watts. Follow also the user manual of the household vacuum cleaner. Do not use vacuum cleaners with higher power.
- Baby-Vac is not a toy. The device must be used by adults after reading the manual.
- Do not use this device for any purpose other than the intended use indicated.
- Do not use this device if any of the components are missing or damaged. Contact your place of purchase for return or replacement information.
- Avoid exposure of this device to extreme temperatures or humidity.

**Storage:**

Store device in a cool, dry environment, out of direct sunlight.

**Assembly:**

- Insert the evacuator (3.) into the collecting cylinder (2.)
- Turn on your household vacuum cleaner.
- Insert the vacuum cleaner adapter (1.) into the attachment hose (Ø2.5-3.8cm) on the vacuum.

**Operation:**

1. Place the child or infant in a sitting/upright position. We recommend to hold the child in your lap until he/she feels comfortable with this apparatus.
2. Position the evacuator (3.) so that it should fully seal one of the nostrils and hold it there for a couple of seconds while observing the discharge being deposited in the collecting cylinder (2.)
3. Repeat the previous step in the same and/or the other nostrils as necessary.
4. You may gently rotate the evacuator in a circular pattern in Steps 2 and 3 to speed up the process.
5. When you are finished, turn off the vacuum and clean the apparatus as described above.
6. Repeat this process daily or as needed.

2012. 10. 01.

Manufacturer Address:  
Illés Csók és Társa  
Ajtósi Dürer sor 35. Budapest H-1146